

# PHOTO-GRID



## Photo-Grid Application:

- Attach Photo-Grid to wall at a height that will allow patient to be photographed with Grid in full background.
- Patient must stand upright in front of the Ortho-Grid.
- Have hair pulled back off face to expose ears and neck.
- Take images with camera at same height as patient—not looking up or down at patient.
- Capture frontal and lateral images of patient with lips at rest and when smiling.

## Additional images may include:

- Bite stick between teeth to document possible maxilla cant.
- Lateral image with stick held against mandibular boarder to document mandibular plain angle (Lateral ceph also recommended).
- Nose/throat openings to document constriction of passages resulting in airway considerations.
- Chin scars to document possible trauma to mandible.
- Maximum Opening and/or Lateral Excursion using **Range of Motion Ruler.**

## Possible observations for treatment planning:

- Head Cant
- Head Posture
- Ear Level
- Eye Level
- Shoulder Level
- Facial Profile
- Lip Angle
- Maxilla Position (soft tissue)
- Mandibular Plain Angle

## Photo-Grid

Please contact the laboratory for pricing and availability.

## Suggested Continuing Education

**Topic: Cranial Distortion**

Dr. Brendan Stack's DVD Series  
**STACK OF KNOWLEDGE**

## **ALTERNATIVE LIGHTWIRE FUNCTIONAL (ALF)**

Drs. Darick Nordstrom, Brendan Stack,  
Gerald Smith and Gavin James

**Rideau Orthodontic Mfg. Ltd**

**(800) 267-7982**

[orthodontic@orthodontic.ca](mailto:orthodontic@orthodontic.ca)

[www.orthodontic.ca](http://www.orthodontic.ca)

E&OE