

PHOTO-GRID



Photo-Grid Application:

- Attach Photo-Grid to wall at a height that will allow patient to be photographed with Grid in full background.
- Patient must stand upright in front of the Ortho-Grid.
- Have hair pulled back off face to expose ears and neck.
- Take images with camera at same height as patient—not looking up or down at patient.
- Capture frontal and lateral images of patient with lips at rest and when smiling.

Additional images may include:

- Bite stick between teeth to document possible maxilla cant.
- Lateral image with stick held against mandibular boarder to document mandibular plain angle (Lateral ceph also recommended).
- Nose/throat openings to document constriction of passages resulting in airway considerations.
- Chin scars to document possible trauma to mandible.
- Maximum Opening and/or Lateral Excursion using **Range of Motion Ruler.**

Possible observations for treatment planning:

- Head Cant
- Head Posture
- Ear Level
- Eye Level
- Shoulder Level
- Facial Profile
- Lip Angle
- Maxilla Position (soft tissue)
- Mandibular Plain Angle

Photo-Grid

SELECT "Photo-Grid Template"
from the CASE EVALUATION MENU OPTION
and print a 24"x 36" poster at your local print shop.

Suggested Continuing Education

Topic: Cranial Distortion

Dr. Brendan Stack's DVD Series
STACK OF KNOWLEDGE

ALTERNATIVE LIGHTWIRE FUNCTIONAL (ALF)

Drs. Darick Nordstrom, Brendan Stack,
Gerald Smith and Gavin James

Rideau Orthodontic Mfg. Ltd

(800) 267-7982

orthodontic@orthodontic.ca

www.orthodontic.ca

E&OE