


# CLASS III FACEMASK

Important Note: To avoid skin irritation, feminine hygienic pads are recommended for lining chin cup and forehead rest. Simply cut the pad to required size and adhere. Replace as required.

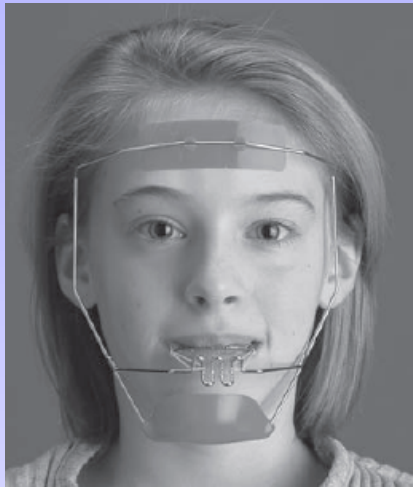

Available in:  
**Small, Medium & Large**  
Colour Supplied: Tan

**Attach Class III bilateral elastics to the lower two hooks on the Facemask.**

## Elastic Force:

As a general rule, elastic force is calculated at 1 ounce/ year of age.

4 years = 4 ounces

6 years = 6 ounces

8 years = 8 ounces

12+ years = 16 ounces

Increase acceptance by starting with a lighter force and increase to required heavier force.

**The facemask is worn at home, watching TV, studying and during sleep as required.**

Case Presentation:  
Class III Treatment utilizing a Class III Sagittal, Fixed Bean Appliance and Class III Facemask.


## Custom Fitting Instructions

### (A) (B) Adjustments and Fitting

To stay in place while sleeping, the face crib must conform closely to contour of forehead and cheeks. Adjustments should be made by applying an adjusting plier at points A and/or B.

\* Adjusting Plier **Bonarch Supply Order #M816**


### (C) Adjust to accommodate a longer or shorter face.

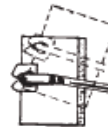



Pad should be above eyebrows for maximum surface contact allowing patient to talk without displacing face crib.

### (D) Prevent appliance from pressing against cheekbone.

### (E) Align mouthbow crib directly with the teeth mouthline. This pressure will hold the face crib more firmly in place.

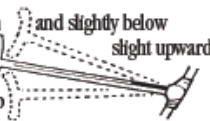

### (F) Slight upward or downward adjustment may be necessary to accommodate individual mandibular position and/or chin musculature.

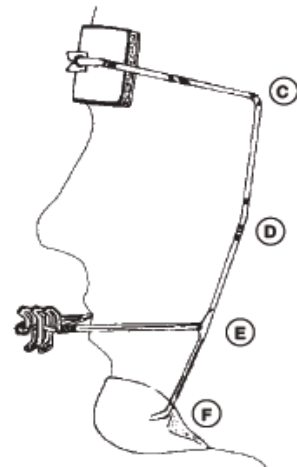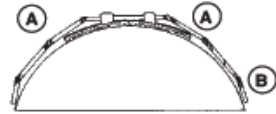

**Due to lightweight stainless steel construction and face-fitting conformity, the necessity of head straps has been eliminated.**

## CLASS III TREATMENT

Dr. Merle Bean a Pediatric Dentist, pioneered class III treatment in North America. Further information is available in his continuing education DVD entitled "Merle's Pearls"

**Rideau Orthodontic Mfg. Ltd**

(800) 267-7982

[orthodontic@orthodontic.ca](mailto:orthodontic@orthodontic.ca)

[www.orthodontic.ca](http://www.orthodontic.ca)

E&OE