

INDIRECT TRIPODING CONSTRUCTION BITE

SAMPLE CONSTRUCTION BITE FOR BALANCED ANTERIOR AND POSTERIOR “INDIRECT” TRIPODING



Rideau Orthodontic Mfg. Ltd.

69 Beckwith Street, North
Smiths Falls, Ontario CANADA

(800) 267-7982

Tel: (613) 283-6841

Fax: (613) 283-5386

www.orthodontic.ca

orthodontic@orthodontic.ca

A PROPER CONSTRUCTION BITE IS ESSENTIAL

*The laboratory will require Mx & Md casts
with the outlined construction bite.*

- **IDEAL CLASS I MOLAR/CUSPID RELATIONSHIP TO A 5% overbite.**
- Take the construction bite with the patient standing upright (not laying back in the dental chair).
- Take a sheet of baseplate wax and cut it in half length wise. Place the wax in a bowl of hot tap water to soften then roll it like a fat pencil. Cut it to the arch length. Place the wax over the lower posterior teeth.
- **DO NOT** cover the incisal of the anterior teeth. Place the wax LINGUAL to the lower incisors so you may visually see the overjet, overbite and midlines.
- **DO NOT** cover the buccal cusps of the molars/bicuspid.
- **The laboratory will** take particular care to mount the patient's casts to the **construction bite**.
- The laboratory **CANNOT PROCEED** with Indirect Tripoding without a construction bite.
- **The laboratory will NEVER ALTER** the construction bite without contacting the doctor.
- **Instructions by the Doctor to alter the construction bite** will be documented and will likely result in considerable additional chair time in order to balance the occlusion.

E&OE 2010

Serving the dental profession since 1972.

Made in Canada

