


NEW ORTHOTIC TWIN BLOCK™

Anti-Snoring • Bruxing • Clenching • TMJD

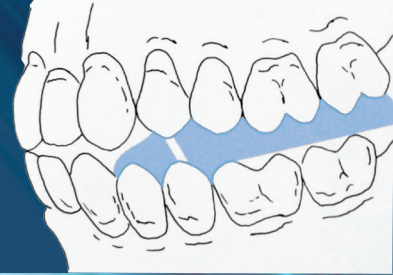

- ✓ Effective
- ✓ Comfortable
- ✓ Affordable
- ✓ Discreet
- ✓ Freedom for Speech
- ✓ Jaw Movement during REM
- ✓ Does not alter the Occlusion
- ✓ Triangular Delta Clasps - optimum retention/minimal fracture

\$270
Shipping
Included!



Select your acrylic colour preference for no additional fee!

Construction Bite (with patient sitting or standing): *The patient's comfort/balance is an important determining factor.*

- An incisal opening of 2-3mm is required to accommodate the lower anterior capping of acrylic
- Mandibular Advancement:**
- Dental/Skeletal Class I:** Generally 3mm of mandibular advancement along with 6-8mm of vertical opening in the lower 2nd bicuspid region.
- Dental/Skeletal Class II:** Advancement of the mandible to an incisal edge to edge relationship along with 6-8mm of vertical opening in the lower 2nd bicuspid region.
- Comparison of a lateral ceph in CR to the construction bite position may allow you to document the improvement to the airway.
- The construction bite position may open the airway sufficiently while dropping the head of the condyles down/forward in their fossae providing healing space for the TM joint/disc. This position could eliminate/reduce SNORING, bruxing /clenching during sleep.



CAUTION: Sleep Apnea can be a life threatening condition. DO NOT attempt to treat a Sleep Apnea patient with any dental oral appliance without instructions by an MD in Sleep Disorders Medicine.



Patients with weak musculature:
Hooks are incorporated to allow placement of 5/16" 2 1/2 ounce inter-arch elastics. The elastics are generally required for the first three months.
The Orthotic Twin Block™ should be worn one hour prior to sleep for the first two weeks for patient acceptance.

TWIN BLOCK™ FUNCTIONAL THERAPY

Applications in Dentofacial Orthopaedics
by Dr. William J Clark, Orthodontist
Professor Dr. T. M. Graber - Chapter 20
"Adult Treatment" Foreword

"It is well known that forward posturing of the mandible has a beneficial effect in most TMD patients."

RIDEAU ORTHODONTIC MFG. LTD.
69 Beckwith Street North • Smiths Falls, ON K7A 2B6
1-800-267-7982 • www.orthodontic.ca • orthodontic@orthodontic.ca
Serving the dental profession since 1972

NEXT DAY Courier service provided by
Purolator
1-888-744-7123
Ship COLLECT under
Acct#3575071