

Ligation made easy ...

Quick >> 2.0® bracket system

Quick bracket and molar bracket.



Quick >> 2.0®



McLaughlin/Bennett/Trevisi-System*

ohne Haken 5-5, mit Haken 6+7
without hook 5-5, with hook 6+7

Slot .018" Patient rechts / Patient right / Paciente derecha / Paziente destra


MAXILLARY

CEFA	731-0713	731-0713	733-0503	733-0403	733-0303	733-0203	733-0103	732-0103	732-0203	732-0303	732-0403	732-0503		730-0713	730-0713	730-0713	730-1603	732-1203	732-1403	732-1503	732-1603		730-1713	730-1713
Torque	-14°	-14°	-7°	-7°	-7°	+10°	+17°	-7°	-7°	-7°	-7°	-7°	-	-14°	-14°	-14°	-7°	0°	0°	2°	2°	-	0°	0°
Angulation	0°	0°	0°	0°	8°	8°	4°	8°	8°	8°	0°	0°	-	0°	0°	0°	0°	3°	3°	2°	2°	-	0°	0°
Torque	-20°	-20°	-12°	-12°	-6°	-6°	-6°	-6°	-6°	-6°	-6°	-6°	-	-20°	-20°	-20°	-17°	-12°	-12°	-17°	-17°	-	-20°	-20°
CEFA	731-1713	731-1713																						


MANDIBULAR

CE 0297

Farbmarkierung / colour-marks / marquage couleur / marcas en los colores siguientes / marcatura colorante

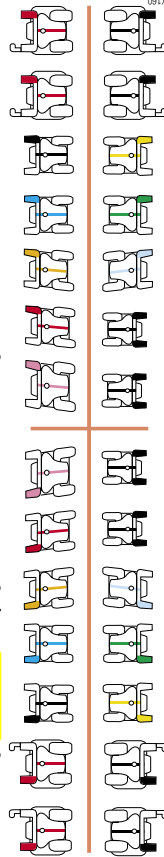

*The FORESTADENT version of these prescriptions are not claimed to be a duplication of any other, nor does Forestadent imply that it is endorsed in any way by the doctor

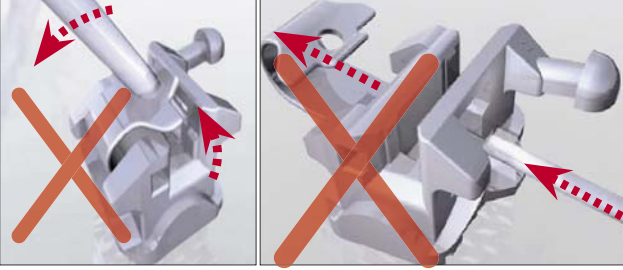

Öffnen Sie die Verschlussklammer nicht durch Aufbiegen oder durch zu große Kraftanwendung.

Do not use a rotational movement or apply excessive force to open the clip, as this may result in loss or breakage.

Ne pas soulever le clip et ne pas appliquer de forces excessives pour ouvrir la barrette.

Non effettuare movimenti rotatori e non applicate forze eccessive ad evitare danni o rotture della clip.

No aplicar fuerzas rotacionales o exceso de fuerza al tratar de abrir el clip ya que puede romperse o despegar el bracket.

Sicherheitshinweis • Safety warning
Avertissement • Precauzioni
Medidas de seguridad



Öffnen und Schließen der Klammer – 2 Optionen


Opening and closing of the bracket – 2 options

Ouverture et fermeture du bracket – 2 options


Apertura e chiusura del bracket – 2 opzioni

Apertura y cierre del bracket – 2 opciones


1a

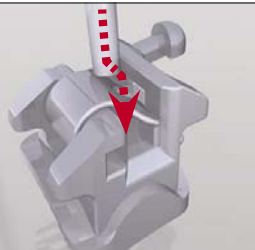


1b

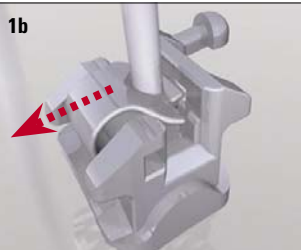

2a

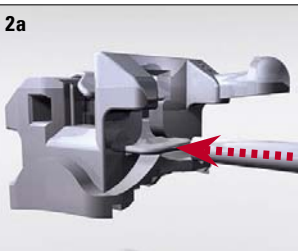

1c/2b

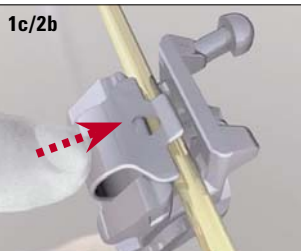



Bogen sorgfältig im Slotgrund halten bevor die Klammer geschlossen wird

Hold archwire firmly down in the slot before closing the clip

Poussez l'arc vers la base de la gorge avant de fermer le clip

Empujar el arco en el slot antes de cerrar el clip

Tenere il arco all' interno dello slot prima di chiudere il clip

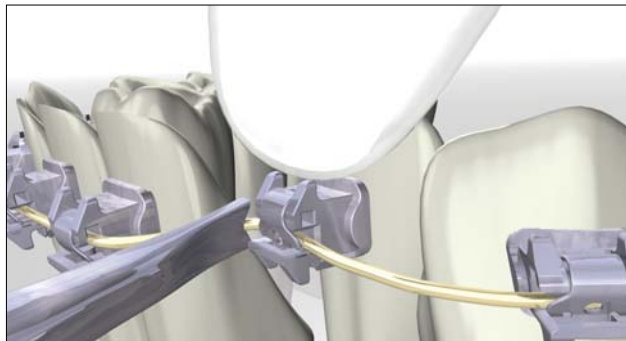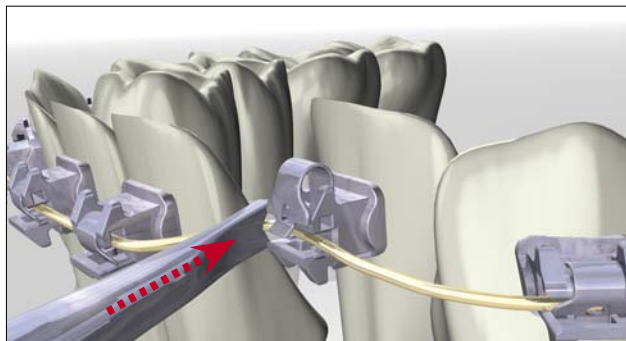

Kleber sparsam dosieren, Bracket an der gingivalen Kante ansetzen

Apply adhesive economically, contact tooth at the gingival aspect first

Appliquez la colle sans excès, placez le bracket d'abord du côté gingival

Aplique el adhesivo de forma no excesiva, posicione el bracket en la parte gingival

Applicare poco adesivo, incollare il bracket dalla zona gengivale



09/2007

CE 0297

Bernhard Förster GmbH

Westliche Karl-Friedrich-Straße 151

75172 Pforzheim - Germany

Tel. + 49 72 31 459-0 · Fax + 49 7231 459-102

info@forestadent.com · www.forestadent.com

FORESTADENT®

www.forestadent.com

Quick-Komfortsonde • Quick-Director • Sondes Quick • Sonda-Quick • Sonda Quick

Order-N° 501-1841